

STEP-BY-STEP REFLEXONIC® PRE-VIBERECT® STRETCH EXERCISES & PROPER APPLICATION OF THE VIBERECT® FOR OPTIMAL SEXUAL RESPONSE



1 Lie on your back or set comfortably.
Grasp the head of penis as demonstrated.



2 Squeeze the head and stretch the penis straight in the air until its natural limit.
You should feel a gentle reflex tightening of your pelvis and penis.



3 Release the stretch back to normal length.
Do not release your grip. Repeat steps 2 & 3 very 2-3 seconds for 2-3 minutes. The penis gradually becomes engorged with arterial blood and gets incrementally rigid, exposing nerve antennas for Viberec stimulation.



4 Hold the Viberec® close to the vibrating end and bring the medical Softpads toward the head of the penis.
Attempt different positions. Standing is a very effective position, with penis located below the heart



5 As you close your grip, the Viberec® automatically activates. Apply pressure.
You can move the Viberec® to find the most sensitive area.



6 Now stretch your penis
While stimulating with the Viberec®, hold for several minutes (5-10 minutes).

Note: As you stimulate, focus on sexual thoughts and intense past memories, or actual sexual encounters in your life. Let both your penis and the brain stimulate sexual centers in your spinal cord. Relax. Avoid impatience and anxiety.

Warning: Use these instructions at your own risk. Never perform pre-Viberec® & Viberec® stimulation if you have sores or tears on your penis. Viberec® use in men with spinal cord injury carries risk of autonomic dysreflexia.



White **£238.8**

£199+ VAT (£238.8) + £8 P&P
100% refund within 60 days - one year warranty

EFFORTLESS OPERATION

especially for men with spinal cord injury to stimulate both sides of the penis with one hand. Safety features for easy turn-on and turn-off with simple grip of the hand.

Cheque Credit Card Debit Card
 Card Valid From _____ Expiry _____ SC _____
 Card Number _____

CERTIFICATIONS:
FDA, CE, ISO 13485:2003, CMDCAS, Health Canada

NAME _____
 ADDRESS _____

 TOWN _____
 COUNTY _____
 POSTCODE _____
 Post to: iMEDicare Ltd, Freepost RTSI-HHAY-GRXC, Unit 11
 Shakespeare Industrial Estate, Shakespeare Street, Watford WD24 5RR

Please visit our website often for updates on exciting multi center clinical trials currently underway on various applications of the Viberec!

The Only Hand Held Medical Device Cleared by FDA and CE certified for the treatment of erectile dysfunction

"There are numerous applications of the Viberec®. It can become a powerful tool in your life in helping to improve and preserve your erectile and sexual health for years to come!"

60 DAY FREE TRIAL

If you're not 100% satisfied, return the device in original condition with all accessories within 60 days for full money back for the purchase price.

How Viberec can help

Viberec safely and painlessly stimulates pudendal nerve receptors located on both sides of the penis that initiate penis erection. Viberec's signals for erection reach spinal cord and "wake up" the sexual centers in sacral spinal cord. Consequently Nitric oxide is released by the cavernous nerves to increase blood flow and erection. Viberec naturally stimulates this process in 3-5 minutes before sexual intercourse or during foreplay.

It also helps contract and strengthen penis and pelvic muscles that are responsible to firmness and maintenance of erection by preventing the blood from leaving the penis. Viberec also amplifies pleasure of orgasm during sexual intercourse.

Viberec is unique in specific frequency and amplitude Vibrations. It is the only vibratory medical device cleared by the US FDA and the European Union for treatment of Erectile Dysfunction (ED). Ideal candidates for Viberec are men with mild to moderate ED, men with unsatisfactory firmness, depression related ED and rehabilitation after prostate cancer treatments and men with spinal cord injury with inability to ejaculate. Viberec can improve response to intracavernosal injection, reduce effective dose and pain.

Distributor UK and ROI -
iMEDicare Ltd
 Freepost RTSI-HHAY-GRXC
 Unit 11 Shakespeare Ind.
 Estate, Shakespeare Str.
 Watford
 WD24 5RR
 Tel 01923 23 7795
 (9am-6pm M-F)
 contact@imedicare.co.uk

Viberec®			
	EMERGO Europe Molenstraat 15, 2513 BH, The Hague The Netherlands +31703458570		
	Manufacturer: Reflexonic, LLC Innovative Urological Solutions 310 Toll House Avenue, Suite 205 Frederick, MD 21701, USA		
	Viberec X3 Viberec Soft Pads		3010

Watch Us on YouTube Under Viberec

NEW REVOLUTIONARY THERAPEUTIC MEDICAL DEVICE FOR HOME & CLINIC USE

VIBERECT®
MADE IN USA

CERTIFIED

www.iMEDicare.co.uk
TO TREAT ERECTILE & EJACULATORY DYSFUNCTION
TEL 01923 23 7795

POWERED BY


WHAT IS VIBERECT®?

Viberect® is a therapeutic vibratory medical device and the only proprietary device in the world that delivers targeted vibratory stimulation to both surfaces of the penis (dorsal and ventral), both branches of the Pudendal nerve, also known by medical and anatomy experts as the "KING OF PELVIS" in both genders. The Viberect® stimulates physiological sexual response by safely utilizing your own pathways of erection and ejaculation.



HOW IT WORKS?

Penile erection is a nerve generated event coordinated by the sacral spinal cord, initiated from excitatory and sexually relevant nerve impulses from the brain and the penis that lead to a complex cascades of events that allow the penis to fill with blood.

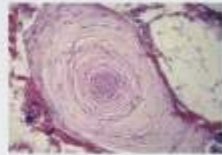

Millions of nerve receptors (antennas) line the surface of the penis. These receptors are especially receptive to repetitive stimulation such as vibration at specific frequency and amplitude provided by the Viberect®.

These nerves communicate with important sexual centers in the brain and spinal cord. They play a critical role in the physiology of sexual response and urinary function.

Sexual arousal (erotic thoughts, sights, genital stimulation, etc.) lead to excitation of sacral spinal cord sexual centers. Their activation leads to a complex cascade of events which cause the penis to fill with blood. In order to achieve firmness, the blood must be prevented from leaking out of the corpora (erection chambers) of the penis.



Many important muscles are strategically located around the base of the penis, which scientists believe are greatly responsible for stabilizing the penis during rigidity phase of erection. They help squeeze the veins to prevent blood from leaving the penis; hence strengthening the rigidity and maintenance of erection. The Viberect strengthens these muscles by persistent activation of Bulbocavernosus Reflex.

For younger men without ED, all nerves, muscles, and vessels responsible for erection are optimized, and they effortlessly play in perfect harmony. As we get older, less healthy, depressed, over-medicated, or have interventions for cancer, then the Nerve/ Vessel/Muscle triad plays out of tune. There is less flow, less excitement, less firmness due to venous leakage, and so on.



POWERFUL HIGH AMPLITUDE & FREQUENCY FOR SCIENTIFIC STIMULATION

- FDA cleared and CE-certified hand-held medical device to provoke penile erection in men who experience erectile dysfunction, and to provoke ejaculation in men with spinal cord injury.
- Developed and recommended by urologists and prostate cancer surgeons to help their patients with erectile dysfunction since 2008.
- Safe, physiological, noninvasive, and powerful tool now available to men around the world to improve and restore their erectile health and sexual response.
- The physiological alternative to drugs, injections, vacuum, and surgery for men with ED.
- Cost effective: Maintenance free, rechargeable batteries, built to last for years with daily use. No extra cost.
- Painless, gentle, pleasant experience. Adjustable frequency and modes for comfort and efficacy.
- Powerful settings to provoke ejaculation in men with spinal cord injury.
- Can be used on-demand before sexual activity, or daily as part of therapy as recommended by your physician.
- Can easily be combined with other modalities (Oral meds, injectables, vacuum, and implants) to improve response in combination for men with severe erectile dysfunction.

3 EASY STEPS TO ORDER

- 1 Call iMEDicare on 01923 23 7795 for more information or to arrange a free home demonstration
- 2 Order online via www.iMEDicare.co.uk (Paypal) or by posting the attached order form
- 3 Dispatch within 3-5 days by Royal Mail First Class Recorded (discrete packaging)

TOP

6 Ways to Prevent Erectile Dysfunction

- 1 Avoid Smoking
- 2 Frequent Aerobic Exercise
- 3 Avoid Triggers of Anxiety & Depression
- 4 Maintain Healthy & Meaningful Relationships
- 5 Eat Healthy
- 6 Keep Your Nerve Reflexes Tuned & Strengthen Your Erectile Muscles With Viberect

Genital Vibratory stimulation is an established modality that has been used for decades for treatment of sexual dysfunction in both men and women.

YOU DESIRE LONG-LASTING ERECTION & RIGIDITY FOR YEARS TO COME?

Erection Is A Reflex. Stimulate Them With Viberect®



Simultaneous stimulation is necessary for optimal sexual response.

Note: Spinal cord injured use carries risk of Autonomic Dysreflexia.